





Shortcuts in DTX Studio™ Clinic 1.9 for Mac

Patient Module



■ Patient Module
■ Image Preview

- Close active patient
- Close preview of the image
- Rotate 2D image clockwise
- Shift + R: Rotate 2D image counterclockwise
- Cmd + R: Reset Zoom
- Flip 2D image horizontally
- Shift + U: Flip 2D image vertically
- Open existing patient in DTX Studio™ Implant
- Shift + P: Export to new patient in DTX Studio™ Implant
- View image in Diagnose
- Open patient in Diagnose
- Capture if no USB sensor is connected
- Capture with last connected sensor device if USB sensor(s) are connected
- Zoom out
- Zoom in
- Prev. image
- Next image
- Shift + Z: Zoom out
- X: Capture
- C: Capture
- B: Bite-wings
- L: Level Window
- >: Export images

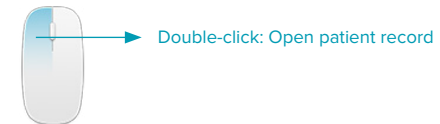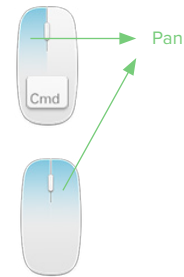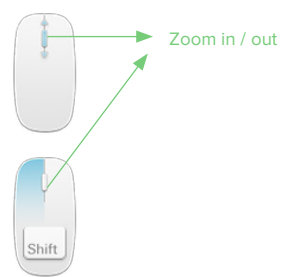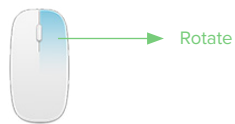

Shortcuts in DTX Studio™ Clinic 1.9 for Mac





Open Patient Record

- Restore down maximized window
- Finish Spotlight action
- Close dialog or wizard

Switch workspaces:

- (fn +) F2: 3D patient / Dental scan
- (fn +) F3: 3D panoramic
- (fn +) F4: Tooth
- (fn +) F5: Panoramic
- (fn +) F6: Intraoral
- (fn +) F7: Ceph
- (fn +) F8: Clinical pictures
- (fn +) F10: Intraoral camera
- (fn +) F12: Recent images

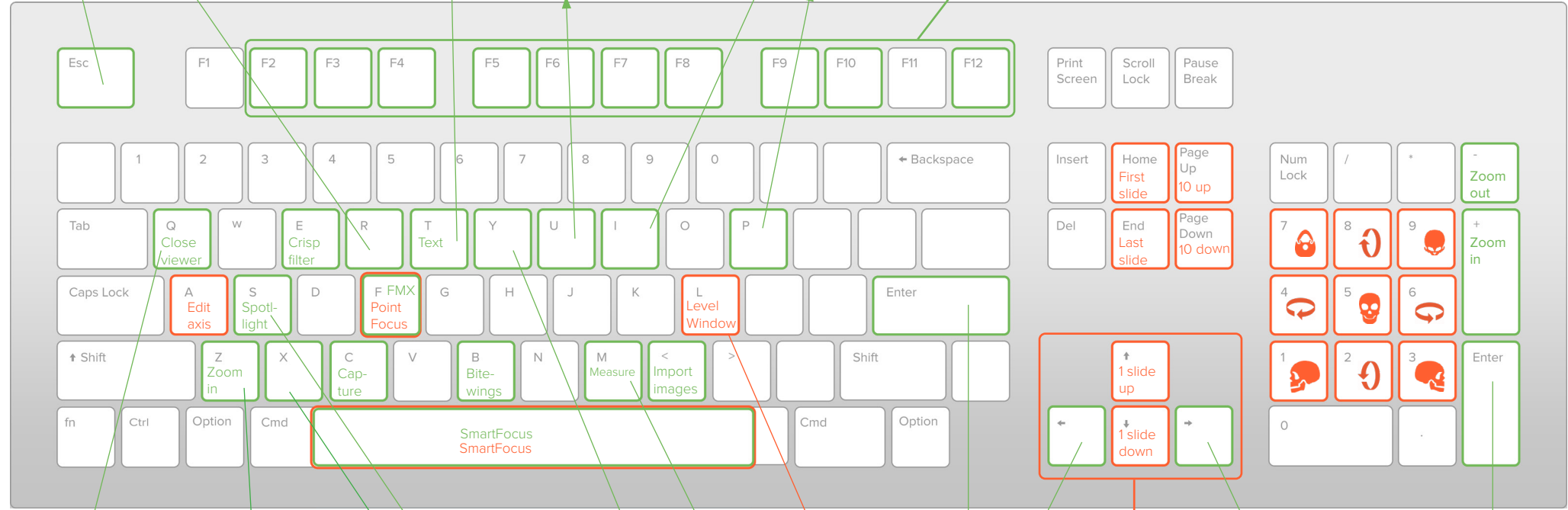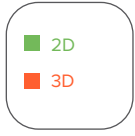

- Rotate image clockwise
- Cmd + R: Reset zoom
- Shift + R: Rotate image counterclockwise

Cmd + T: Adjust tooth range

- Flip image horizontally
- Shift + U: Flip image vertically

Cmd + I: Show / Hide viewer info

Cmd + P: Print viewer images and content

Esc

F1 F2 F3 F4 F5 F6 F7 F8 F9 F10 F11 F12

Print Screen Scroll Lock Pause Break

1 2 3 4 5 6 7 8 9 0 ← Backspace

Insert Home Page Up First slide 10 up

Num Lock / * - Zoom out

Tab Q Close viewer W E Crisp filter R T Text Y U I O P

Del End Last slide Page Down 10 down

7 8 9 + Zoom in

Caps Lock A Edit axis S Spotlight D F FMX Point Focus G H J K L Level Window Enter

↑ Shift Z Zoom in X C Capture V B Bite-wings N M Measure < Import images > Shift

↑ 1 slide up ↓ 1 slide down

4 5 6 Enter

fn Ctrl Option Cmd SmartFocus SmartFocus Cmd Option

1 2 3 Enter

Cmd + Q: Quit application

- Cmd + Z: Undo
- Cmd + Shift + Z: Redo
- Shift + Z: Zoom out

Cmd + S: Save (diagnosis)

Capture with a connected sensor

Cmd + Y: Redo

Cmd + M: Maximize / Restore down

Adjust brightness and contrast

Navigate to previous maximized image

Navigate to next maximized image

Rotate 3D view

Capture with intraoral camera (when the livestream is active)

